

i-PRF Tubes 10ml Instructions

This device is a single use product. It should not be re-used.
 This blood collection tube is use for i-PRF™ preparation.
 The tube is sterile, plain vacuum, without additives or anti-coagulants.
 In a single package.

i-PRF
 CHOUKROUN



Procedure:

- Take of the tube from its packaging.
- Make sure of the integrity of the package. If the package is opened or damaged, please throw it and take an intact product.
- Plug the tube on the holder.
- The tube is filled and stops automatically when is full (vacuum tubes). Capacity: 10 ml.
- Unplug the tubes and go to next. When the desired number of tubes is reached, place them in the centrifuge.
- Tubes are ready for centrifugation. To set RPM and time of centrifugation, please contact us.

We will give you the settings adapted to your centrifuge.


NB: When the procedure i-PRF is completed, please immediately put the used tubes in the containers provided for this purpose.

Precautions:

- The proper functioning of this device is assured only if the instructions are followed.
- Don't do any action that may affect the device;
- Check the integrity of the individual before using package, if the package is damaged, do not use.
- In case of exposure during an injury or needle stick, seek medical advice.
- The re-use of the product is strictly prohibited; it can lead to dangerous infections to health, or even death.
- In the case of hemophilia patients, epileptics etc., special attention is needed during the drawing.



1. Take the blood collection tubes i-PRF™.

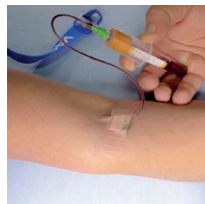


2. Make the blood drawing, fill the required number of tubes.




3. Place the tubes in the centrifuge

LOT Lot number

 Manufacturing date

 Single use product
Should not be re-used

CE

 Expiry date : Product can be used until the end of the month

STERILE R Gamma sterilization

Non toxic - Non pyrogenic